

**Directions des Opérations
Réglementaires et Pharmaceutiques**

Information de sécurité

stérilisation

Etablissement

**Pharmacien responsable du DM
Pharmacien Responsable**

Adresse

CP - Ville

Saint-Cloud, le 7 juillet 2020

**Information de sécurité
Instruments chirurgicaux pour système nerveux central
(Voir liste jointe en annexe 2)**

Madame, Monsieur,

Le fabricant B. Braun Aesculap AG a déterminé qu'un certain nombre de ses instruments chirurgicaux peut être utilisé au niveau du système nerveux central. Les instruments concernés par cette information de sécurité sont listés dans l'annexe 2(a/b).

Le lavage manuel / désinfection tel qu'indiqué dans les instructions d'utilisation actuellement disponibles ne s'applique plus aux applications de neurochirurgie.

Conformément à la réglementation en vigueur, nous vous informons que ces instruments doivent faire l'objet d'un retraitement spécifique précisé comme suit

Lavage mécanique/ désinfection avec pré-nettoyage manuel

Partie1 – Pré-nettoyage manuel.

Applicable aux pinces bipolaires et aux canules d'aspiration/tubes d'aspiration de dissection

Pré-nettoyage manuel avec brosse

Note

Applicable uniquement aux canules d'aspiration et pinces bipolaires

Phase	Phase	T (°C/F)	t (min)	Conc (%)	Qualité de l'eau	Chimie
I	Lavage	TA (froid)	>15	1	EP	B.Braun Stabimed® fresh
II	Rinçage	TA (froid)	1	-	EP	-

EP: eau potable

TA: Température ambiante

Phase I

- ▶ Nettoyer l'instrument dans la solution de lavage à l'aide d'une brosse adaptée jusqu'à ce que tous les résidus visibles soient éliminés de la surface.
- ▶ Si applicable, brosser les surfaces non visibles avec une brosse appropriée pendant au moins 1 min.
- ▶ Au cours du lavage, actionner les composants mobiles tels que les vis de réglages, les articulations, etc.
- ▶ Rincer soigneusement ces composants avec la solution lavante désinfectante (au moins 5 fois) à l'aide d'une seringue à usage unique (20 ml).

Phase II

- ▶ Rincer l'instrument soigneusement (toutes les surfaces accessibles) sous l'eau courante.
- ▶ Au cours du rinçage, actionner les composants mobiles tels que les vis de réglages, les articulations, etc..

Partie 2 –Lavage mécanique / Désinfection

Applicable à tous les instruments listés en annexe 2a, incluant ceux ayant fait l'objet d'un pré-nettoyage manuel décrit dans la partie 1.

Lavage alcalin en machine et désinfection thermique

Type de machine: appareil de lavage/désinfection à une chambre sans ultrason ¹⁾

Phase	Etape	T (°C/F)	t (min)	Conc (%)	Qualité de l'eau	Chimie
I	Rinçage préalable	<25/77	3		EP	-
II	Nettoyage	55/131	10		EDém	Dr Weigert neodisher® Septoclean – solution 1% ²⁾
III	Neutralisation	>10/50	2		EDém	B.Braun Helimatic® Neutralizer C Solution 0,15%
IV	Rinçage intermédiaire I	>10/50	1		EDém	-
V	Rinçage intermédiaire II	>10/50	1		EDém	-
VI	Désinfection thermique	90/194	5		EDém	-
VII	Séchage	-	-	-	-	Selon le programme de nettoyage et de désinfection de l'appareil

EP: eau potable

EDém: eau déminéralisée

1) Dispositif de lavage et désinfection utilisé pour contrôler la nettoyabilité: Miele 7836 CD

2) Agent pronicide (voir informations techniques Dr. Weigert neodisher® SeptoClean)

- ▶ Placer l'instrument dans un panier adapté au lavage (éviter les angles morts).
- ▶ Placer l'instrument dans un panier avec tous ces composants et articulations en position ouverte
- ▶ Raccorder directement les pièces avec lumières et canaux au raccord d'irrigation spécial du chariot à injection.
- ▶ Contrôler la présence éventuelle de résidus sur les surfaces visibles des instruments après le lavage mécanique/ désinfection

Stérilisation

Applicable à tous les dispositifs mentionnés en annexe 2a

- ▶ **Veiller à ce que la vapeur ait accès à toutes les surfaces internes et externes (p. ex. en ouvrant les valves et les robinets).**
- ▶ **Procédé de stérilisation validé**
 - Stérilisation à la vapeur avec procédé du vide fractionné
 - Stérilisateur à la vapeur selon la norme DIN EN 285 et validé selon la norme DIN EN ISO 17665

– **Stérilisation par procédé du vide fractionné à 134 °C, pendant 18 min pour inactivation du prion.**

► **En cas de stérilisation simultanée de plusieurs produits dans un stérilisateur à vapeur: veiller à ce que la charge maximale autorisée du stérilisateur à vapeur indiquée par le fabricant ne soit pas dépassée.**

Les validations internes des instruments concernés confirment l'adéquation et l'applicabilité des procédures spécifiées. Il n'y a donc pas de risque patient accru.

Nous vous demandons **d'accuser réception** de l'information de sécurité par retour du document joint en annexe 1 (A.R).

L'ANSM a été informée de cette information de sécurité.

Pour toute information complémentaire, vous pouvez prendre directement contact avec le Service Vigilance Qualité Produits de B. Braun Médical - tél : 01-41-10-74-84.

Restant à votre disposition, nous vous prions d'agréer, Madame, Monsieur, nos salutations distinguées.

Manuelle SCHNEIDER-PONSOT
Directeur des Opérations Réglementaires et Pharmaceutiques
Correspondant matériovigilant
Directeur Général

**Directions des Opérations
Réglementaires et Pharmaceutiques**

Annexe 1

Information de sécurité – Accusé de réception
instruments chirurgicaux pour système nerveux central

Confirmation de réception par :

fax au : 01-70-83-45-06

e-mail : vigilance_rappel.fr@bbraun.com

**Nous avons pris note de l'information de sécurité concernant les instruments
chirurgicaux pour système nerveux central.**

Les utilisateurs / services ont été informés des mesures à mettre en place.

Date	Nom/Fonction
Signature	

Nom de l'établissement (Tampon de l'établissement):

Etablissement - CP Ville

Code client:

Annexe 2a: Liste des instruments retraits concernés

Référence	Libellé
FD481B	MICRO SCISSORS CVD.BL/SH 140MM
FD482B	MICRO SCISSORS CVD.SH/SH 140MM
FD483B	MICRO SCISSORS CVD.SH/SH 170MM
FD484B	MICRO SCISSORS STR.SH/SH WL:100MM 225MM
FD485B	MICRO SCISSORS CVD.BL/SH WL:100MM 225MM
FD488B	MICRO FORCEPS STR.0.3MM 140MM
FD489B	MICRO FORCEPS STR.0.3MM 170MM
FD490B	MICRO FORCEPS STR.0.3MM 210MM
FD493B	MICRO NEEDLE HOLDER STR.0.5MM 150MM
FD494B	MICRO NEEDLE HOLDER CVD.0.5MM 170MM
FD495B	MICRO NEEDLE HOLDER CVD.0.5MM 210MM
FD510B	MICRO FORCEPS RND HANDLE BAY.45DG 210MM
FD511B	MICRO FORCEPS RND HANDLE BAY.90DG 210MM
FF341R	TOENNIS TUMOR GRASPING FORCEPS SMALL
FF342R	TOENNIS TUMOR GRASPING FORCEPS MEDIUM
FF343R	TOENNIS TUMOR GRASPING FORCEPS LARGE
GF005B	NOIR THUMB CONTROL MODULAR
GF025B	NOIR SUCTION TUBE 4FR 80MM
GF026B	NOIR SUCTION TUBE 6FR 80MM
GF027B	NOIR SUCTION TUBE 8FR 80MM
GF030B	NOIR SUCTION TUBE LE.CVD.6FR 100MM
GF031B	NOIR SUCTION TUBE RI.CVD.6FR 100MM
GF032B	NOIR SUCTION TUBE LE.CVD.8FR 100MM
GF033B	NOIR SUCTION TUBE RI.CVD.8FR 100MM
GF035B	NOIR SUCTION TUBE 4FR 100MM
GF036B	NOIR SUCTION TUBE 6FR 100MM
GF037B	NOIR SUCTION TUBE 8FR 100MM
GF038B	NOIR SUCTION TUBE W/HOLES 6FR 100MM
GF039B	NOIR SUCTION TUBE W/HOLES 8FR 100MM
GF045B	NOIR SUCTION TUBE 4FR 120MM
GF046B	NOIR SUCTION TUBE 6FR 120MM
GF047B	NOIR SUCTION TUBE 8FR 120MM
GF048B	NOIR SUCTION TUBE W/HOLES 6FR 120MM
GF049B	NOIR SUCTION TUBE W/HOLES 8FR 120MM
GF055B	NOIR SUCTION TUBE 4FR 140MM
GF056B	NOIR SUCTION TUBE 6FR 140MM
GF057B	NOIR SUCTION TUBE 8FR 140MM
GF058B	NOIR SUCTION TUBE W/HOLES 6FR 140MM
GF059B	NOIR SUCTION TUBE W/HOLES 8FR 140MM
GF235R	THUMB PLATE MODULAR SUCTIONS TEAR DROP
GF240R	SUCTION ATTM.MALLEAB.TAPERED 3FR 100MM
GF241R	SUCTION ATTM.MALLEAB.TAPERED 4FR 100MM

Référence	Libellé
GF242R	SUCTION ATTM.MALLEAB.TAPERED 5FR 100MM
GF243R	SUCTION ATTM.MALLEAB.TAPERED 6FR 100MM
GF244R	SUCTION ATTM.MALLEAB.TAPERED 7FR 100MM
GF245R	SUCTION ATTM.MALLEAB.TAPERED 8FR 100MM
GF246R	SUCTION ATTM.MALLEAB.TAPERED 9FR 100MM
GF247R	SUCTION ATTM.MALLEAB.TAPERED 10FR 100MM
GF248R	SUCTION ATTM.MALLEAB.TAPERED 12FR 100MM
GF250R	SUCTION ATTM.MALLEAB.TAPERED 3FR 115MM
GF251R	SUCTION ATTM.MALLEAB.TAPERED 4FR 115MM
GF252R	SUCTION ATTM.MALLEAB.TAPERED 5FR 115MM
GF253R	SUCTION ATTM.MALLEAB.TAPERED 6FR 115MM
GF254R	SUCTION ATTM.MALLEAB.TAPERED 7FR 115MM
GF255R	SUCTION ATTM.MALLEAB.TAPERED 8FR 115MM
GF256R	SUCTION ATTM.MALLEAB.TAPERED 9FR 115MM
GF257R	SUCTION ATTM.MALLEAB.TAPERED 10FR 115MM
GF258R	SUCTION ATTM.MALLEAB.TAPERED 12FR 115MM
GF260R	SUCTION ATTM.MALLEAB.TAPERED 3FR 140MM
GF261R	SUCTION ATTM.MALLEAB.TAPERED 4FR 140MM
GF262R	SUCTION ATTM.MALLEAB.TAPERED 5FR 140MM
GF263R	SUCTION ATTM.MALLEAB.TAPERED 6FR 140MM
GF264R	SUCTION ATTM.MALLEAB.TAPERED 7FR 140MM
GF265R	SUCTION ATTM.MALLEAB.TAPERED 8FR 140MM
GF266R	SUCTION ATTM.MALLEAB.TAPERED 9FR 140MM
GF267R	SUCTION ATTM.MALLEAB.TAPERED 10FR 140MM
GF268R	SUCTION ATTM.MALLEAB.TAPERED 12FR 140MM
GF270R	SUCTION ATTM.MALLEAB.TAPERED 3FR 165MM
GF271R	SUCTION ATTM.MALLEAB.TAPERED 4FR 165MM
GF272R	SUCTION ATTM.MALLEAB.TAPERED 5FR 165MM
GF273R	SUCTION ATTM.MALLEAB.TAPERED 6FR 165MM
GF274R	SUCTION ATTM.MALLEAB.TAPERED 7FR 165MM
GF275R	SUCTION ATTM.MALLEAB.TAPERED 8FR 165MM
GF276R	SUCTION ATTM.MALLEAB.TAPERED 9FR 165MM
GF277R	SUCTION ATTM.MALLEAB.TAPERED 10FR 165MM
GF278R	SUCTION ATTM.MALLEAB.TAPERED 12FR 165MM
GF350R	FERGUSSON SUCT.CANN D:1.5MM WORK.L.110MM
GF352R	FERG-FRAZIER SUCT 7FR110MM WRK-LGT
GF354R	FERG-FRAZIER SUCT 12FR 110MM WRK-LGT
GF355R	FERGUSSON SUCT.CANN D:5.0MM WORK.L.110MM
GF360R	FERGUSSON SUCT.CANN D:1.5MM WORK.L.130MM
GF361R	FERGUSSON SUCT.CANN D:2.0MM WORK.L.130MM
GF362R	FERGUSSON SUCT.CANN D:2.5MM WORK.L.130MM
GF363R	FERGUSSON SUCT.CANN D:3.0MM WORK.L.130MM
GF364R	FERGUSSON SUCT.CANN D:4.0MM WORK.L.130MM
GF365R	FERGUSSON SUCTION CANNULA 5MM 130MM

Référence	Libellé
GF370R	FERGUSSON SUCT.CANN D:1.5MM WORK.L.160MM
GF371R	FERGUSSON SUCT.CANN D:2.0MM WORK.L.160MM
GF372R	FERGUSSON SUCT.CANN D:2.5MM WORK.L.160MM
GF373R	FERGUSSON SUCT.CANN D:3.0MM WORK.L.160MM
GF374R	FERGUSSON SUCT.CANN D:4.0MM WORK.L.160MM
GF375R	FERGUSSON SUCT.CANN D:5.0MM WORK.L.160MM
GF380R	FERGUSSON SUCT.CANN D:1.5MM WORK.L.200MM
GF381R	FERGUSSON SUCT.CANN D:2.0MM WORK.L.200MM
GF382R	FERGUSSON SUCT.CANN D:2.5MM WORK.L.200MM
GF383R	FERGUSSON SUCT.CANN D:3.0MM WORK.L.200MM
GF384R	FERGUSSON SUCT.CANN D:4.0MM WORK.L.200MM
GF385R	FERGUSSON SUCT.CANN D:5.0MM WORK.L.200MM
GF391R	SUCTION TUBE TAPERED TEARDROP 3FR 180MM
GF392R	SUCTION TUBE TAPERED TEARDROP 4FR 180MM
GF393R	SUCTION TUBE TAPERED TEARDROP 5FR 180MM
GF394R	SUCTION TUBE TAPERED TEARDROP 6FR 180MM
GF395R	SUCTION TUBE TAPERED TEARDROP 7FR 180MM
GF396R	SUCTION TUBE TAPERED TEARDROP 8FR 180MM
GF397R	SUCTION TUBE TAPERED TEARDROP 9FR 180MM
GF398R	SUCTION TUBE TAPERED TEARDROP 10FR 180MM
GF399R	SUCTION TUBE TAPERED TEARDROP 12FR 180MM
GF401R	SUCTION TUBE TAPERED TEARDROP 3FR 165MM
GF402R	SUCTION TUBE TAPERED TEARDROP 4FR 165MM
GF403R	SUCTION TUBE TAPERED TEARDROP 5FR 165MM
GF404R	SUCTION TUBE TAPERED TEARDROP 6FR 165MM
GF405R	SUCTION TUBE TAPERED TEARDROP 7FR 165MM
GF406R	SUCTION TUBE TAPERED TEARDROP 8FR 165MM
GF407R	SUCTION TUBE TAPERED TEARDROP 9FR 165MM
GF408R	SUCTION TUBE TAPERED TEARDROP 10FR 165MM
GF409R	SUCTION TUBE TAPERED TEARDROP 12FR 165MM
GF411R	SUCTION TUBE TAPERED TEARDROP 3FR 205MM
GF412R	SUCTION TUBE TAPERED TEARDROP 4FR 205MM
GF413R	SUCTION TUBE TAPERED TEARDROP 5FR 205MM
GF414R	SUCTION TUBE TAPERED TEARDROP 6FR 205MM
GF415R	SUCTION TUBE TAPERED TEARDROP 7FR 205MM
GF416R	SUCTION TUBE TAPERED TEARDROP 8FR 205MM
GF417R	SUCTION TUBE TAPERED TEARDROP 9FR 205MM
GF418R	SUCTION TUBE TAPERED TEARDROP 10FR 205MM
GF419R	SUCTION TUBE TAPERED TEARDROP 12FR 205MM
GF421R	SUCTION TUBE TAPERED TEARDROP 3FR 230MM
GF422R	SUCTION TUBE TAPERED TEARDROP 4FR 230MM
GF423R	SUCTION TUBE TAPERED TEARDROP 5FR 230MM
GF424R	SUCTION TUBE TAPERED TEARDROP 6FR 230MM
GF425R	SUCTION TUBE TAPERED TEARDROP 7FR 230MM

Référence	Libellé
GF426R	SUCTION TUBE TAPERED TEARDROP 8FR 230MM
GF427R	SUCTION TUBE TAPERED TEARDROP 9FR 230MM
GF428R	SUCTION TUBE TAPERED TEARDROP 10FR 230MM
GF429R	SUCTION TUBE TAPERED TEARDROP 12FR 230MM
GF470R	BENDING RESIST.MICRO SUCTION 4FR 80MM
GF471R	BENDING RESIST.MICRO SUCTION 6FR 80MM
GF472R	BENDING RESIST.MICRO SUCTION 8FR 80MM
GF473R	BENDING RESIST.MICRO SUCTION 4FR 100MM
GF474R	BENDING RESIST.MICRO SUCTION 6FR 100MM
GF475R	BENDING RESIST.MICRO SUCTION 8FR 100MM
GF476R	BENDING RESIST.MICRO SUCTION 4FR 120MM
GF477R	BENDING RESIST.MICRO SUCTION 6FR 120MM
GF478R	BENDING RESIST.MICRO SUCTION 8FR 120MM
GF479R	BENDING RESIST.MICRO SUCTION 4FR 140MM
GF480R	BENDING RESIST.MICRO SUCTION 6FR 140MM
GF481R	BENDING RESIST.MICRO SUCTION 8FR 140MM
GF923R	FRAZIER SUCTLUER-HUB9FR100MMWRK-LGTH
GF925R	FRAZIER SUCTLUER-HUB15FR5/100MM LGTH
GF980R	LANDOLT DISS SUCTION TUBE 3.2MM DIA
GK595R	BIPOL.FORCEPS AAG FL.STR.0.6MM 120MM
GK596R	BIPOL.FORCEPS AAG FL.STR.0.3MM 145MM
GK597R	BIPOL.FORCEPS AAG FL.CVD.0.6MM 110MM
GK604R	BIPOL.FORCEPS AAG FL.STR.0.9MM 160MM
GK605R	BIPOL.FORCEPS AAG FL.STR.0.9MM 185MM
GK606R	BIPOL.FORCEPS AAG FL.STR.0.9MM 200MM
GK614R	BIPOL.FORCEPS AAG FL.BAYO.F.0.7MM 160MM
GK615R	BIPOL.FORCEPS AAG FL.BAYO.F.1.0MM 185MM
GK616R	BIPOL.FORCEPS AAG FL.BAYO.F.1.0MM 200MM
GK624R	BIPOL.FORCEPS AAG FL.ANGL.0.9MM 160MM
GK625R	BIPOL.FORCEPS AAG FL.ANGL.0.8MM 200MM
GK644R	PRECIOUS METAL FORCEPS160MM 0.7MM
GK645R	PRECIOUS METAL FORCEPS 185MM 1.0MM
GK646R	PRECIOUS METAL FORCEPS 200MM 1.0MM
GK660R	BIPOLAR COAGULATION FCPS 0.6MM 160MM
GK670R	BIPOLAR COAG.FORCEPS 0.6MM 200MM
GK671R	BIPOLAR FORCEPS INS.BAYO.9MM 200MM
GK675R	BIPOLAR COAGULATION FORCEPS.6MM220MM
GK677R	BIPOLAR COAGULATION FORCEPS.6MM 240MM
GK679R	BIPOLAR COAGULATION FORCEPS 0.9MM240MM
GK690R	BIPOLAR COAGULATION FORCEPS0.4MMX120MM
GK700R	COAGULATION FORCEPS INSUL.160MM STR.
GK701R	COAGULATION FCPS STR INS 160MM
GK703R	COAGUL.FORCEPS INSULATED 200MM STR.
GK711R	BIPOLAR COAGULATION FCPSIND.9MMX160MM

Référence	Libellé
GK713R	COAG.FORC.INCUL 200MMWORK.PART ANGL
GK720R	BIPOLAR COAGULATION FORCEPS .6MMX160MM
GK722R	BIPOLAR COAGULATION FORCEPS .9MMX160MM
GK723R	BIPOLAR COAGULATION FCPSANG.9MMX160MM
GK730R	BIPOLAR COAGULATION FORCEPS 0.9MM200MM
GK743R	BIPOL.RING FCPS.RING D:5.0 WORK.L.75MM
GK746R	YASARGIL BIPOLAR FORCEPS 0.4X20/140MM
GK747R	YASARGIL BIPOLAR FORCEPS 0.7X20/140MM
GK748R	YASARGIL BIPOLAR FORCEPS 1.0X20/140MM
GK749R	YASARGIL BIPOLAR FORCEPS 1.3X20/140MM
GK760R	YASARGIL BIPOLAR FORCEPS 0.4MMX35/155MM
GK761R	YASARGIL BIPOLAR FORCEPS 0.7X35/155MM
GK762R	YASARGIL BIPOLAR FORCEPS 1.0X35/155MM
GK763R	YASARGIL BIPOLAR FORCEPS 1.3X35/155MM
GK764R	YASARGIL BIPOLAR FORCEPS 0.4X55/175MM
GK765R	YASARGIL BIPOLAR FORCEPS 0.7X55/175MM
GK766R	YASARGIL BIPOLAR FORCEPS 1.0X55/175MM
GK767R	YASARGIL BIPOLAR FORCEPS 1.3X55/175MM
GK768R	YASARGIL BIPOLAR FORCEPS 0.4X75/195MM
GK769R	YASARGIL BIPOLAR FORCEPS 0.7X75/195MM
GK770R	YASARGIL BIPOLAR FORCEPS 1.0X75/195MM
GK771R	YASARGIL BIPOLAR FORCEPS 1.3X75/195MM
GK772R	YASARGIL BIPOLAR FORCEPS 0.4X95/215MM
GK773R	YASARGIL BIPOLAR FORCEPS 0.7X95/215MM
GK774R	YASARGIL BIPOLAR FORCEPS 1.0X95/215MM
GK775R	YASARGIL BIPOLAR FORCEPS 1.3X95/215MM
GK777R	YASARGIL BIPOLAR FCP45DG-UP7X95/215MM
GK780R	YASARGIL BIPOLAR FCP90DG-UP.4X95/215MM
GK781R	YASARGIL BIPOLAR FCP90DG-UP.7X95/215MM
GK785R	YASARGIL BIPOLR FCP90DG-DWN.7X95/215MM
GK789R	YASARGIL BIPOLAR FORCEPS 0.7X115/235MM
GK790R	YASARGIL BIPOLAR FORCEPS 1.0X115/235MM
GK791R	YASARGIL BIPOLAR FORCEPS 1.3X115/235MM
GK793R	YASARGIL BIPOLAR FORCEPS 0.7X135/255MM
GK807R	BIPOLAR COAGULATION FORCEPS 175MM
GK808R	BIPOLAR COAGULATION FORCEPS 220MM
GK809R	BIPOLAR COAGULATION FORCEPS 125MM
GK822R	BIPOL.FORCEPS STR.BAY.0.7MM 95MM
GK823R	BIPOL.FORCEPS 45°BAY.0.7MM 95MM
GK824R	BIPOL.FORCEPS STR.BAY.1.0MM 95MM
GK825R	BIPOL.FORCEPS 45°BAY.1.0MM 95MM
GK826R	BIPOL.FORCEPS STR.BAY.0.7MM 135MM
GK827R	BIPOL.FORCEPS 45°BAY.0.7MM 135MM
GK828R	BIPOL.FORCEPS STR.BAY.1.0MM 135MM

Référence	Libellé
GK829R	BIPOL.FORCEPS 45°BAY.1.0MM 135MM
GK831R	BIPOLAR FORCEPS RING-TIP 2.3X235MM
GK832R	BIPOLAR FORCEPS RING-TIP 3.4X235MM
GK833R	BIPOLAR FORCEPS RING-TIP 4.5X235MM
GK838R	BIPOL.FCPS.W/SANO IRRIGATION 165MM 0.7MM
GK840R	BIPOLAR FRCPS W/IRRIG CHANNEL.4MM180MM
GK841R	BIPOLAR FRCPS W/IRRIG CHANNEL.7MM180MM
GK842R	IRRIGATION FORCEPS 180MM 1.3MM TIP
GK843R	BIPOLAR FRCP W/IRRIG CHANNEL 0.4MM205MM
GK844R	BIPOLAR FRCP W/IRRIG CHANNEL .7MM205MM
GK845R	IRRIGATION FORCEPS 205MM 1.3MM TIP
GK846R	BIPOLAR FRCP W/IRRIG CHANNEL .4MM230MM
GK847R	BIPOLAR FRCP W/IRRIG CHANNEL.7MM230MM
GK848R	IRRIGATION FORCEPS 230MM1.3MM TIP
GK871R	BIP.RING IRRIG.FORCEPS RING D:5MM 235MM
GK899R	BIPOLAR COAGULATION FORCEPS 165/0.2MM
GK900R	CASPAR BIPOLAR FORCEPS 0.5X70/165MM
GK910R	CASPAR BIPOLAR FORCEPS 1.0X70/165MM
GK920R	CASPAR BIPOLAR FORCEPS 2.0X70/165MM
GK929R	BIPOLAR COAGULATION FORCEPS 195/0.2MM
GK930R	CASPAR BIPOLAR FORCEPS 0.5X100/195MM
GK940R	CASPAR BIPOLAR FORCEPS 1.0X100/195MM
GK950R	CASPAR BIPOLAR FORCEPS 2.0X125/195MM
GK959R	BIPOLAR COAGULATION FORCEPS 220/0.2MM
GK960R	CASPAR BIPOLAR FORCEPS 0.5X125/220MM
GK970R	CASPAR BIPOLAR FORCEPS 1.0X125/220MM
GK972R	GKSPAR BIPOLAR FCPS45DG-UP1.X125/220MM
GK974R	CASPAR BIPOLAR FCP45DG-DWN1.X125/220MM
GK976R	CASPAR BIPOLAR FCP90DG-UP 1.X120/215MM
GK978R	CASPAR BIPOLAR FCP90DG-DWN1.X120/215MM
GK980R	CASPAR BIPOLAR FORCEPS 2.0X125/220MM
GL470R	BIPROTECT PARTLY INSUL.TAB CON.195/0.4MM
GL471R	BIPROTECT PARTLY INSUL.TAB CON.195/0.8MM
GL480R	BIPROTECT PARTLY INSUL.TAB CON.215/0.4MM
GL481R	BIPROTECT PARTLY INSUL.TAB CON.215/0.8MM
S61017	SUCTION CANNULA
S67507	COAG. FORCEPS
US161	GOLDTIP BIPOLAR FCPS STR 220/95/0.5MM
US162	GOLDTIP BIPOLAR FCPS STR 220/95/1.0MM
US163	GOLDTIP BIPOLAR FCPS STR 220/95/1.5MM
US164	GOLDTIP BIPOLAR FCPS CVD-UP 220/95/0.5MM
US171	GOLDTIP BIPOLAR FCPS STR 125/1.0MM
US172	GOLDTIP BIPOLAR FCPS STR 100/0.4MM
US173	GOLDTIP BIPOLAR FCPS STR 255/150/1.5MM

Référence	Libellé
US182	CUSHING BIPOLAR FCPSBAYO,INS,1.0MM190MM
US183	CUSHING BIPOLAR FCPSBAYO,INS,1.5MM190MM
US184	HARDY BIPOLAR FORCEPS INS 1.0MM215MM
US185	HARDY BIPOLAR FORCEPS INS 1.5MM215MM
US186	HARDY BIPOLAR FORCEPS INS 2.0MM215MM
US187	HARDY BIPOLAR FORCEPS INS 0.5MM255MM
US188	HARDY BIPOLAR FORCEPS INS 1.0MM255MM
US189	HARDY BIPOLAR FORCEPS INS 1.5MM255MM
US305	CUSHING BIPOLAR FCPSBAYOINS2MM190MM
US308	CUSHING BIPOLAR FCPSBAYOINS0.7MM190MM
US310	HARDY BIPOLAR FORCEPS INS .5MM215MM
US403	GOLDTIP BIPOLAR FCPS STR 150/50/0.5MM
US404	GOLDTIP BIPOLAR FCPS STR 150/50/0.7MM
US405	GOLDTIP BIPOLAR FCPS STR 150/50/1.0MM
US406	GOLDTIP BIPOLAR FCPS STR 150/50/1.5MM
US407	GOLDTIP BIPOLAR FCPS STR 150/50/2.0MM
US486	GOLDTIP BIPOLAR FCPS STR 150/50/0.25MM
US487	GOLDTIP BIPOLAR FCPS STR 170/70/0.25MM
US488	GOLDTIP BIPOLAR FCPS STR 170/70/0.5MM
US489	GOLDTIP BIPOLAR FCPS STR 170/70/0.7MM
US490	GOLDTIP BIPOLAR FCPS STR 170/70/1.0MM
US491	GOLDTIP BIPOLAR FCPS STR 170/70/1.5MM
US492	GOLDTIP BIPOLAR FCPS STR 170/70/2.0MM
US493	GOLDTIP BIPOLAR FCPS STR 190/90/0.25MM
US494	GOLDTIP BIPOLAR FCPS STR 215/115/0.25MM
US495	GOLDTIP BIPOLAR FCPS STR 230/125/0.25MM
US496	GOLDTIP BIPOLAR FCPS STR 255/150/0.25MM
US530	GOLDTIP BIPOLAR FCPS STR 255/150/0.5MM
US531	GOLDTIP BIPOLAR FCPS STR 255/150/2.0MM
US600	GOLDTIP BIPOLAR FCPS STR 190/90/0.5MM
US601	GOLDTIP BIPOLAR FCPS STR 190/90/0.7MM
US602	GOLDTIP BIPOLAR FCPS STR 190/90/1.0MM
US603	GOLDTIP BIPOLAR FCPS STR 190/90/1.5MM
US604	GOLDTIP BIPOLAR FCPS STR 190/90/2.0MM
US610	GOLDTIP BIPOLAR FCPS STR 215/115/0.5MM
US611	GOLDTIP BIPOLAR FCPS STR 215/115/0.7MM
US612	GOLDTIP BIPOLAR FCPS STR 215/115/1.0MM
US613	GOLDTIP BIPOLAR FCPS STR 215/115/1.5MM
US614	GOLDTIP BIPOLAR FCPS STR 215/115/2.0MM
US615	GOLDTIP BIPOLAR FCPS STR 230/125/0.5MM
US616	GOLDTIP BIPOLAR FCPS STR 230/125/0.7MM
US617	GOLDTIP BIPOLAR FCPS STR 230/125/1.0MM
US618	GOLDTIP BIPOLAR FCPS STR 230/125/1.5MM
US619	GOLDTIP BIPOLAR FCPS STR 230/125/2.0MM

Référence	Libellé
US620	GOLDTIP BIPOLAR FCPS STR 255/150/0.7MM
US621	GOLDTIP BIPOLAR FCPS STR 255/150/1.0MM
XG601	WFNS BASIC SET OF BIPOLAR COAGULATION

**Annexe 2b: Liste des instruments à usage unique concernés
(Non retraitables / non stérilisables)**

Référence	Libellé
GK501SU	GOLDTIP DISP.BIPOLAR FCPS 190/90/0,5MM
GK502SU	GOLDTIP DISP.BIPOLAR FCPS 190/90/1,0MM
GK503SU	GOLDTIP DISP.BIPOLAR FCPS 190/90/1,5MM
GK504SU	GOLDTIP DISP.BIPOLAR FCPS 215/115/0,5MM
GK505SU	GOLDTIP DISP.BIPOLAR FCPS 215/115/1,0MM
GK506SU	GOLDTIP DISP.BIPOLAR FCPS 215/115/1,5MM
GK507SU	GOLDTIP DISP.BIPOLAR FCPS 190/90/0.25MM
GK508SU	GOLDTIP DISP.BIPOLAR FCPS 215/115/0.25MM
GK509SU	GOLDTIP DISP.BIPOLAR FCPS 190/90/0.7MM
GK510SU	GOLDTIP DISP.BIPOLAR FCPS 215/115/0.7MM