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Installation and Closing Confirmation Sheet

IMAGEnet6

Name of customer Address/Telephone number	Date of check (yyyy/mm/dd):	
	Agency/Supplier	Business office/Checker

• Work information

Date of work (yyyy/mm/dd)	Company name: Worker name:		
Check items (Check the following items according to the working procedure.)			
No.	Content of work	Checking point	Check
1	The software name and the version (It should be "IMAGEnet6: Ver 1.24 or less.)	Check if the software name and the version is in question. IMAGEnet6 : Ver.	
2	The serial number confirmation	Check the serial number of IMAGEnet 6 IMAGEnet6 : S/N.	
3	IMAGEnet 6 installation	Confirm if any error message is displayed during the installation	
4	Operation check	After turning on, confirm the Login, display of patients list, and Logout operation can be done normally.	
5	New software version (It should be: Ver 1.24.1 or more)	Check new software version IMAGEnet6 : Ver.1.24.1	
6	Confirm with customer	Does the customer have the IMAGEnet 6 DVD version 1.24 or lower? In case of yes, collect DVD seperately and confirm other sheet.	
7		The customer is aware and understands the FSN	

Total result	Type of support	Date of confirmation	Signature of worker
OK · NG	On-site or Remote	(yyyy/mm/dd)	

Check by customer

<input checked="" type="checkbox"/> We have checked that the installation procedure is completed.
Name and signature from Customer Representative

1. Make PDF file of this sheet and send it to TEM (fsn-ima6@topcon.eu) 2. Copy for customer 3. Keep original sheet yourself

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Topcon Corporation
Topcon Europe Medical B.V.