**Date :** 06 Sep 23

**Reference :** REC687

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**Annexe A**

**Rx Daytona Plus:**

The screenshot shows the 'System Setup 1' window for the Radox Daytona Plus. The interface includes a top navigation bar with icons for various functions and a left sidebar with icons for different system components. The main area is divided into several sections:

- Host Communication Mode:** Radio buttons for 'Off Line', 'On Line Batch', and 'On Line Real time'.
- Host Details Mode:** Radio buttons for 'Auto rerun', 'Confirm host for rerun', and 'Inquire host for rerun'.
- Host Communications:** Fields for Baud Rate (9600), Data Bit (8bit), Stop Bit (1bit), Parity Bit (Even), and Protocol (E13B1).
- Others:** A list of checkboxes including 'Disable extrapolated results', 'Auto scan for Reagent Container', 'Check remaining reagent volume', 'Lamp voltage control off', 'Initial Result of Rerun', and 'Auto Rerun' (which is circled in red).
- Sample:** Fields for Sample number increment (Numeric only), Upper sample number, and Digit (15).
- Alarm sound:** Checkboxes for 'Enable Sound for sampling completion', 'Enable Error Level', 'Alarm Volume', and 'Round Completion Alarm'.
- Reagent Code for Reagent Pipette Wash:** A dropdown menu.
- Reagent code for Cuvette Water Placement:** A dropdown menu.
- Remaining test calculation:** Radio buttons for 'Reagent Code' and 'Reagent Lot'.
- Live Print out:** Checkboxes for 'Enable' and 'Enable Live QC Report'.
- Printout Header:** A table with 5 rows for header information.

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