

# **URGENT : AVIS DE SÉCURITÉ**

**Kits de canules de trachéotomie BLUselect®, Kits de canules de trachéotomie BLUselect® Suctionaid®, Kit de procédure de trachéotomie par dilatation percutanée BLUgriggs® ou plateau avec canule de trachéotomie BLUselect® avec ou sans pinces, Plateau de procédure de dilatation BLUperc® avec produits de dilatation en une seule étape, Procédure de trachéotomie par dilatation percutanée BLUperc® Kit ou plateau avec ou sans tube de trachéotomie BLUselect®**


19 août 2024

Chers clients BLUSelect ®,

Smiths Medical, Inc. publie cet avis de sécurité urgent pour vous informer d'un défaut potentiel avec les produits BLUSelect ®, BLUgriggs ® et BLUperc ® répertoriés dans *l'annexe 1\_Produits concernés*. Cette lettre détaille le problème et les étapes à suivre.

## **Problème:**

Smiths Medical a identifié la possibilité d'une déconnexion du ballonnet témoin de la ligne de gonflage du ballonnet de trachéotomie dans des lots spécifiques de produits BLUSelect ®, BLUgriggs ® et BLUperc ® en raison d'un défaut de fabrication. Voir l'exemple du défaut potentiel sur la photographie ci-dessous.

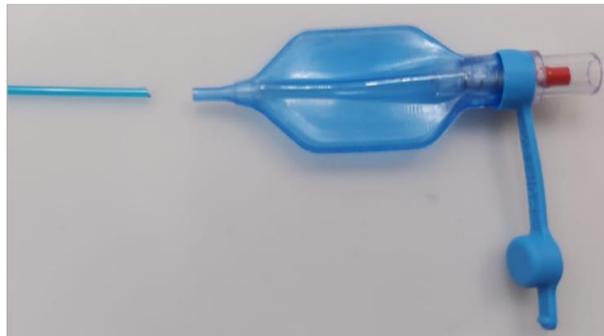

## **Risque potentiel :**

Si le ballonnet témoin utilisé pour gonfler le ballonnet de trachéotomie se détache de la ligne de gonflage, la pression du ballonnet peut ne pas être maintenue, ce qui peut entraîner une ventilation inadéquate et un risque accru d'aspiration des sécrétions pour le patient. À ce jour, Smiths Medical a reçu treize (13) rapports faisant état de blessures graves et de zéro (0) décès associés à ce problème.

## **Produit concerné**

Veuillez consulter les numéros de produits et de lot concernés dans la pièce jointe *1\_Liste des produits concernés* ainsi que la plage de dates de distribution des produits.

## **Actions médicales Smiths :**

Smiths Medical envoie cette notification à tous les clients BLUSelect ®, BLUgriggs ® et BLUperc ® qui ont reçu des produits de Smiths Medical répertoriés dans *l'annexe 1\_Produit concerné*. Smiths Medical accordera un crédit aux clients concernés dès réception d'un formulaire de réponse complété confirmant la destruction du produit.

**Actions requises par le client :**

Lors de l'utilisation des dispositifs, toutes les instructions, y compris les avertissements et les précautions contenues dans la documentation du mode d'emploi, doivent être suivies avec une vigilance accrue. Veuillez effectuer les actions suivantes répertoriées ci-dessous

1. Vérifiez tous les emplacements de stockage au sein de votre établissement pour identifier les produits et les numéros de lot concernés répertoriés dans la notification et cessez leur utilisation. Jetez tous les produits concernés en suivant le processus de mise au rebut de votre établissement. Si l'élimination n'est pas immédiatement possible dans votre établissement, le produit doit alors être mis en quarantaine jusqu'à son élimination.
2. Partagez cette notification avec tous les utilisateurs potentiels des dispositifs pour vous assurer qu'ils sont au courant de cette notification et des mesures d'atténuation proposées. Si les appareils sont utilisés à un autre endroit, veuillez-vous assurer que cette communication y est transmise.
3. Remplissez et renvoyez le formulaire de réponse client ci-joint à [EMEA-FSN@icumed.com](mailto:EMEA-FSN@icumed.com) dans les 10 jours suivant sa réception pour accuser réception de cette notification.
4. **DISTRIBUTEURS** : Si vous avez distribué des produits potentiellement concernés à vos clients, veuillez leur transmettre immédiatement cet avis et leur demander de remplir le formulaire de réponse et de **VOUS le retourner**. Ensuite, le **DISTRIBUTEUR** doit remplir un SEUL formulaire avec les détails requis et le retourner à [EMEA-FSN@icumed.com](mailto:EMEA-FSN@icumed.com)

Pour plus d'informations, veuillez contacter Smiths Medical en utilisant les informations suivantes :

Contact médical Smiths	Coordonnées	Domaines de soutien
Gestion globale des réclamations	<a href="mailto:globalcomplaints@icumed.com">globalcomplaints@icumed.com</a>	Pour signaler des événements indésirables ou des plaintes concernant un produit
Service client	<a href="https://www.icumed.com/about-us/contact-us">https://www.icumed.com/about-us/contact-us</a>	Questions sur le remplacement du produit et/ou le crédit.

L'autorité compétente nationale a été informée de cette action.

Smiths Medical s'engage envers la sécurité des patients et s'efforce de fournir une fiabilité exceptionnelle des produits et le plus haut niveau de satisfaction client. Merci pour votre soutien rapide sur cette notification importante. Nous apprécions votre coopération.

Sincèrement,

Andy Mathein  
Vice-président de la qualité

**Fichiers attachés :**

- Formulaire de réponse client (voir ci-dessous)
- Liste des produits concernés (pièce jointe 1)

## AVIS DE SÉCURITÉ URGENT - FORMULAIRE DE RÉPONSE

**Kits de canules de trachéotomie BLUselect®, Kits de canules de trachéotomie BLUselect® SuctionAid®, Kit de procédure de trachéotomie par dilatation percutanée BLUgriggs® ou plateau avec canule de trachéotomie BLUselect® avec ou sans pinces, Plateau de procédure de dilatation BLUperc® avec produits de dilatation en une seule étape, Procédure de trachéotomie par dilatation percutanée BLUperc® Kit ou plateau avec ou sans tube de trachéotomie BLUselect®**

19 août 2024

**Vérifiez votre inventaire et complétez les informations ci-dessous, même si vous ne possédez pas le produit concerné. Le fait de ne pas remplir toutes les sections de cette page peut entraîner un avoir incorrect, retardé ou refusé.**

Veillez renvoyer le formulaire complété à [EMEA-FSN@icumed.com](mailto:EMEA-FSN@icumed.com). Si vous avez des questions sur ce formulaire, veuillez contacter [EMEA-FSN@icumed.com](mailto:EMEA-FSN@icumed.com) ou votre représentant commercial local.

Nom de l'hôpital/établissement	
Adresse de l'hôpital/établissement	
Numéro de téléphone	
Nom et titre de la personne remplissant ce formulaire	
Signature de la personne remplissant ce formulaire	
Date	
En cas d'achat auprès d'un distributeur, veuillez indiquer ici le nom/l'emplacement du distributeur à des fins de traçabilité.	

**Cochez une des cases ci-dessous :**

- Je **n'ai AUCUN** produit concerné (remplissez et renvoyez ce formulaire à l'adresse e-mail ci-dessus)
- OUI**, j'ai des produits concernés, j'ai informé les utilisateurs de mon établissement et j'ai suivi les instructions qui m'ont été fournies et détruit tous les articles concernés (voir tableau ci-dessous)

**Si vous avez des produits concernés en stock, veuillez compléter le tableau 1 ci-dessous :**

**TABLEAU 1**

Numéro d'article/SKU	Numéro de lot	Quantité en inventaire (Chacun)	Quantité détruite (unités)	Date de destruction

**Si vous avez distribué des produits concernés, veuillez compléter le tableau 2 ci-dessous avec les informations rassemblées reçues de vos clients et répondre à ICU Medical avec les informations globales.**

**TABLEAU 2**

Numéro d'article/SKU	Numéro de lot	Quantité détruite localement (unités)	Date de destruction

**Les événements indésirables et les plaintes associés à l'utilisation de ce produit doivent être signalés et envoyés par courrier électronique au service mondial de gestion des plaintes de Smiths Medical à [globalcomplaints@icumed.com](mailto:globalcomplaints@icumed.com).**







101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4348849
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4348850
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4348859
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4348860
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4350945
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4385043
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4385046
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4385047
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4385048
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4385049
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4385050
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4396535
101/563/080	BLUperc® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube	4396536
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	3913653
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	3925768
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4115435
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4156962
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4212065
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4238178
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4281031
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4348865
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4348866
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4386559
101/563/090	BLUperc® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube	4408641
101/800/060CZ	BLUSELECT 6.0, CUFFED, NON-FEN	4230149
101/800/060CZ	BLUSELECT 6.0, CUFFED, NON-FEN	4382002
101/800/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4037975
101/800/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4249341
101/800/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4305944
101/800/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4368641
101/800/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4348898
101/800/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4365892
101/800/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4052621
101/800/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4138225
101/800/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4305933
101/800/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4305934
101/800/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4305935
101/800/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4305936
101/800/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	4348861
101/800/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4306047
101/800/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4306058
101/800/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4372161
101/800/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4386600
101/800/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4390744
101/802/060CZ	BLUSELECT 6.0, CUFFED, FEN	4028452
101/802/060CZ	BLUSELECT 6.0, CUFFED, FEN	4382046
101/802/075CZ	BLUSELECT 7.5, CUFFED, FEN	3968265
101/802/075CZ	BLUSELECT 7.5, CUFFED, FEN	4388729
101/802/080CZ	BLUSELECT 8.0, CUFFED, FEN	3951185
101/802/080CZ	BLUSELECT 8.0, CUFFED, FEN	4268672
101/802/080CZ	BLUSELECT 8.0, CUFFED, FEN	4310441
101/802/080CZ	BLUSELECT 8.0, CUFFED, FEN	4423828
101/802/080CZ	BLUSELECT 8.0, CUFFED, FEN	4436040
101/802/080CZ	BLUSELECT 8.0, CUFFED, FEN	4456953
101/802/085CZ	BLUSELECT 8.5, CUFFED, FEN	4028454
101/802/090CZ	BLUSELECT 9.0, CUFFED, FEN	3925752
101/802/090CZ	BLUSELECT 9.0, CUFFED, FEN	3954847
101/810/060CZ	BLUSELECT 6.0, CUFFED, NON-FEN	3887495
101/810/060CZ	BLUSELECT 6.0, CUFFED, NON-FEN	3933190
101/810/060CZ	BLUSELECT 6.0, CUFFED, NON-FEN	3958315
101/810/060CZ	BLUSELECT 6.0, CUFFED, NON-FEN	4106722
101/810/060CZ	BLUSELECT 6.0, CUFFED, NON-FEN	4268673

101/810/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	3887478
101/810/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4106813
101/810/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4243097
101/810/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4254571
101/810/070CZ	BLUSELECT 7.0, CUFFED, NON-FEN	4285856
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	3887504
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	3925795
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4010055
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4073763
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4177908
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4222083
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4249233
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4281153
101/810/075CZ	BLUSELECT 7.5, CUFFED, NON-FEN	4381994
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	3887481
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4173265
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4207825
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4217114
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4230143
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4285773
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4306016
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4312272
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4381980
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4387365
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4403461
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4415817
101/810/080CZ	BLUSELECT 8.0, CUFFED, NON-FEN	4423832
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	3922028
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	3951211
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	3989280
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	4067915
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	4098973
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	4152435
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	4178022
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	4281079
101/810/085CZ	BLUSELECT 8.5, CUFFED, NON-FEN	4306011
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	3887482
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	3993623
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4106736
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4173266
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4243117
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4306027
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4327669
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4403367
101/810/090CZ	BLUSELECT 9.0, CUFFED, NON-FEN	4458561
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	3909242
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4217875
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4249333
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4291208
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4310443
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4312263
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4387368
101/810/100CZ	BLUSELECT 10.0, CUFFED, NON-FEN	4397557
101/812/060CZ	BLUSELECT 6.0, CUFFED, FEN	3887479
101/812/060CZ	BLUSELECT 6.0, CUFFED, FEN	3925779
101/812/060CZ	BLUSELECT 6.0, CUFFED, FEN	3959080
101/812/060CZ	BLUSELECT 6.0, CUFFED, FEN	3976752
101/812/060CZ	BLUSELECT 6.0, CUFFED, FEN	4106752
101/812/060CZ	BLUSELECT 6.0, CUFFED, FEN	4294444
101/812/070CZ	BLUSELECT 7.0, CUFFED, FEN	3887499
101/812/070CZ	BLUSELECT 7.0, CUFFED, FEN	3925781
101/812/070CZ	BLUSELECT 7.0, CUFFED, FEN	3968279

101/812/070CZ	BLUSELECT 7.0, CUFFED, FEN	4106754
101/812/070CZ	BLUSELECT 7.0, CUFFED, FEN	4242910
101/812/075CZ	BLUSELECT 7.5, CUFFED, FEN	3887552
101/812/075CZ	BLUSELECT 7.5, CUFFED, FEN	3968280
101/812/075CZ	BLUSELECT 7.5, CUFFED, FEN	4217831
101/812/075CZ	BLUSELECT 7.5, CUFFED, FEN	4268677
101/812/075CZ	BLUSELECT 7.5, CUFFED, FEN	4381979
101/812/075CZ	BLUSELECT 7.5, CUFFED, FEN	4413594
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	3887488
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	3948295
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	3989255
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	4073770
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	4106756
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	4227246
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	4234757
101/812/080CZ	BLUSELECT 8.0, CUFFED, FEN	4385065
101/812/085CZ	BLUSELECT 8.5, CUFFED, FEN	3922030
101/812/085CZ	BLUSELECT 8.5, CUFFED, FEN	3925793
101/812/085CZ	BLUSELECT 8.5, CUFFED, FEN	4028504
101/812/085CZ	BLUSELECT 8.5, CUFFED, FEN	4191164
101/812/085CZ	BLUSELECT 8.5, CUFFED, FEN	4222085
101/812/085CZ	BLUSELECT 8.5, CUFFED, FEN	4261649
101/812/085CZ	BLUSELECT 8.5, CUFFED, FEN	4382043
101/812/090CZ	BLUSELECT 9.0, CUFFED, FEN	3972311
101/812/090CZ	BLUSELECT 9.0, CUFFED, FEN	4106757
101/812/090CZ	BLUSELECT 9.0, CUFFED, FEN	4170983
101/812/090CZ	BLUSELECT 9.0, CUFFED, FEN	4217120
101/812/090CZ	BLUSELECT 9.0, CUFFED, FEN	4243118
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	3948187
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4064797
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4245460
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4259283
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4381995
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4388732
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4413618
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4423838
101/812/100CZ	BLUSELECT 10.0, CUFFED, FEN	4443290
101/860/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	3909675
101/860/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	4065369
101/860/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	4386514
101/860/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4357891
101/860/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4405274
101/860/090CZ	BLUSELECT 9.0, SUCTIONAID, CUFFED, NON-FEN	4310533
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	3951204
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	3968404
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	4178001
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	4217826
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	4249291
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	4256336
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	4283719
101/870/070CZ	BLUSELECT 7.0, SUCTIONAID, CUFFED, NON-FEN	4291171
101/870/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	3951207
101/870/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	4249236
101/870/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	4270941
101/870/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	4276631
101/870/075CZ	BLUSELECT 7.5, SUCTIONAID, CUFFED, NON-FEN	4382010
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	3989180
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4017250
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4051928
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4054796
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4070522
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4134866

101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4163106
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4208997
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4238168
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4254564
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4303380
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4310523
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4312285
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4317581
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4319800
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4368642
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4388736
101/870/080CZ	BLUSELECT 8.0, SUCTIONAID, CUFFED, NON-FEN	4390747
101/870/085CZ	BLUSELECT 8.5, SUCTIONAID, CUFFED, NON-FEN	3925788
101/870/085CZ	BLUSELECT 8.5, SUCTIONAID, CUFFED, NON-FEN	4002986
101/870/085CZ	BLUSELECT 8.5, SUCTIONAID, CUFFED, NON-FEN	4085217
101/870/085CZ	BLUSELECT 8.5, SUCTIONAID, CUFFED, NON-FEN	4183981
101/870/085CZ	BLUSELECT 8.5, SUCTIONAID, CUFFED, NON-FEN	4212070
101/870/085CZ	BLUSELECT 8.5, SUCTIONAID, CUFFED, NON-FEN	4249238
101/870/085CZ	BLUSELECT 8.5, SUCTIONAID, CUFFED, NON-FEN	4376585
101/870/090CZ	BLUSELECT 9.0, SUCTIONAID, CUFFED, NON-FEN	3892136
101/870/090CZ	BLUSELECT 9.0, SUCTIONAID, CUFFED, NON-FEN	4249334
101/870/090CZ	BLUSELECT 9.0, SUCTIONAID, CUFFED, NON-FEN	4285763
101/870/090CZ	BLUSELECT 9.0, SUCTIONAID, CUFFED, NON-FEN	4285764
101/870/090CZ	BLUSELECT 9.0, SUCTIONAID, CUFFED, NON-FEN	4303381
101/870/090CZ	BLUSELECT 9.0, SUCTIONAID, CUFFED, NON-FEN	4387367
101/870/100CZ	BLUSELECT 10.0, SUCTIONAID, CUFFED, NON-FEN	4217806
101/870/100CZ	BLUSELECT 10.0, SUCTIONAID, CUFFED, NON-FEN	4256343
101/870/100CZ	BLUSELECT 10.0, SUCTIONAID, CUFFED, NON-FEN	4306010
101/891/070	BLUgriggs® Percutaneous Dilatation Procedural Kit with forceps and 7.0mm BLUselect® Suctionaid® Trach Tube	3925816
101/891/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with forceps and 8.0mm BLUselect® Suctionaid® Trach Tube	3887523
101/891/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with forceps and 8.0mm BLUselect® Suctionaid® Trach Tube	3896997
101/891/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with forceps and 8.0mm BLUselect® Suctionaid® Trach Tube	3896998
101/891/090	BLUgriggs® Percutaneous Dilatation Procedural Kit with forceps and 9.0mm BLUselect® Suctionaid® Trach Tube	4005342
101/893/070	BLUgriggs® Percutaneous Dilatation Procedural Kit with 7.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4005284
101/893/070	BLUgriggs® Percutaneous Dilatation Procedural Kit with 7.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4089723
101/893/070	BLUgriggs® Percutaneous Dilatation Procedural Kit with 7.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4281033
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4010052
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4070524
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4238251
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4281154
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4285878
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4310417
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4382056
101/893/080	BLUgriggs® Percutaneous Dilatation Procedural Kit with 8.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4382057
101/893/090	BLUgriggs® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube, no forceps	3925810
101/893/090	BLUgriggs® Percutaneous Dilatation Procedural Kit with 9.0mm BLUselect® Suctionaid® Trach Tube, no forceps	4310431