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ADDENDUM TO THE USER MANUAL FOR SIMEOX_H

Update on contraindications and expected side effects

Contra-indications

Absolute contra-indications are:

- Neuromuscular disease with respiratory muscle weakness
- Inability to cough properly and independently
- Actively requiring assistance with inspiration (e.g., ventilatory assistance with endotracheal tube, tracheostomy tube, or ventilated with a mask)

Relative contra-indications are:

- Active or recent gross hemoptysis
- Recent cardiothoracic surgery, including esophageal surgery
- Severe acute lung injury
- Acute pneumothorax
- Inspiratory muscle weakness with inability to tolerate increased work of breathing
- Severe restrictive disease (Total Lung Capacity <60%)
- Oropharyngeal and/or oral muscle weakness
- Uncontrolled gastroesophageal reflux disease (GERD)
- Airway at risk for aspiration such as recent tube feeding or meal
- Unstable cardiovascular pathologies (recent heart attack, unstable angina, uncontrolled arrhythmias, unstable heart failure)
- Hemodynamic instability

Expected side effects

- Uncommon expected side effects during and/or following SIMEOX_H are:
 1. of mild intensity and transient: hemoptysis, excessive sputum, chest discomfort or pain, chest pain with sputum, excessive cough, dyspnea or shortness of breath
 2. of moderate intensity: aggravation of pre-existing hemoptysis
- Rare expected side effects during and/or following SIMEOX_H are:
 1. of mild intensity and transient: dry throat, hoarseness, throat irritation, laryngitis, bronchi irritation, gastric reflux
 2. of moderate intensity: nausea and/or vomiting in pregnant women, aggravation of gastroesophageal reflux, sore throat, low back pain, arrhythmias in patients with a history of rhythm disorders or with a pacemaker

If the patient experiences one or more side effects while using SIMEOX_H, it is recommended to consult a healthcare professional as soon as possible to assess whether treatment should be continued.

- If the patient is taking anti-coagulants and is prone to hemoptysis, it is recommended to consult a healthcare professional for advice before using SIMEOX_H, as the risk of an increase in pre-existing hemoptysis may be greater in some cases.
- In the event of dry throat during SIMEOX_H treatment, it is recommended to hydrate sufficiently (by drinking water, for example) between bronchial drainage sessions to avoid hoarseness or irritation.
- Regular hydration also reduces the risk of bronchial irritation. Inhalations can also help to humify the mucous membranes of the throat and bronchi.
- SIMEOX_H should be used away from meals, to avoid any discomfort or potential digestive upset (nausea, gastric reflux, vomiting, etc.), particularly in pregnant women and patients with a history of gastro-esophageal reflux disease.

- It is recommended to use SIMEOX_H in a comfortable, relaxed position, to avoid inappropriate muscular and joint tension during treatment, which could induce pain.
- SIMEOX_H should not cause significant pain or discomfort to the patient: in the event of significant pain or discomfort, it is recommended to stop using SIMEOX_H immediately and consult a healthcare professional.
- In the event of a new unexpected symptom or worsening of an existing symptom, it is recommended to stop using SIMEOX_H immediately and consult a healthcare professional.