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## **AirSeal®** Access Port

Bladeless Optical Tip, Low Profile Bladeless Optical Tip,  
Palm Grip Bladeless Optical Tip, and Blunt Tip Access Ports

## **AirSeal®** Filtered Tube Sets

Tri-Lumen Filtered Tube Set with Activated Charcoal Filter,  
Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter,  
Single Lumen Filtered Tube Set

For use with **AirSeal®** iFS

FIGURES .....	2
ENGLISH .....	4
FRENCH.....	9
ITALIAN.....	13
GERMAN .....	18
SPANISH .....	23
GREEK .....	27
PORTUGUESE .....	32
PORTUGUESE (BRAZILIAN) .....	37
DUTCH .....	41
DANISH.....	46
SWEDISH.....	50
FINNISH.....	54
NORWEGIAN.....	59
CHINESE.....	63
JAPANESE .....	67
POLISH .....	72
TURKISH.....	77
ROMANIAN .....	81
ARABIC.....	86
RUSSIAN .....	90
CROATIAN .....	95
CZECH .....	99
HUNGARIAN .....	104
LITHUANIAN .....	108
SLOVAK.....	113
SLOVENIAN .....	117
KOREAN .....	122
BULGARIAN .....	126

<p>Fig. 1</p>	
<p>Fig. 2, 3, 4, 5</p>	
<p>Fig. 6</p>	
<p>Fig. 7</p>	

Fig. 8

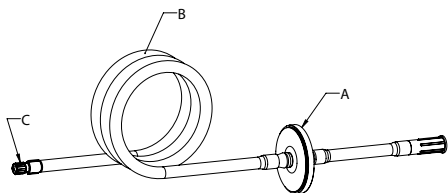
Fig. 9

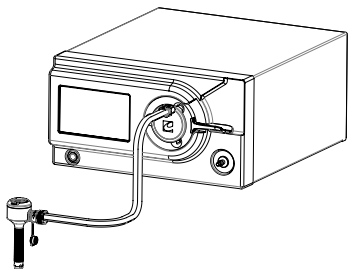
Fig. 10

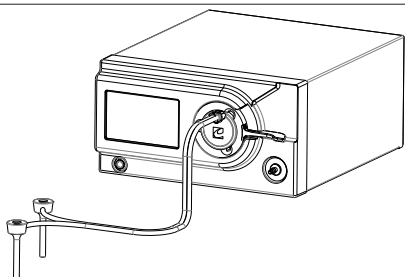
Fig. 11



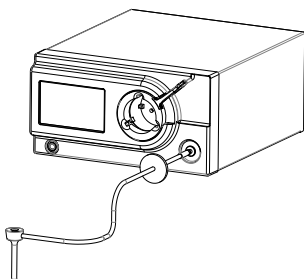
Fig. 12

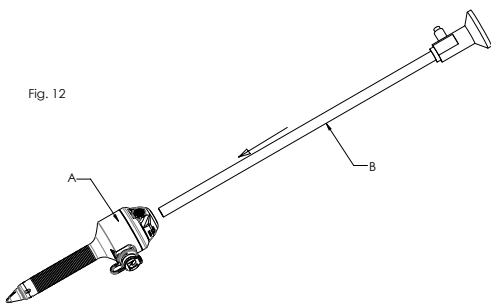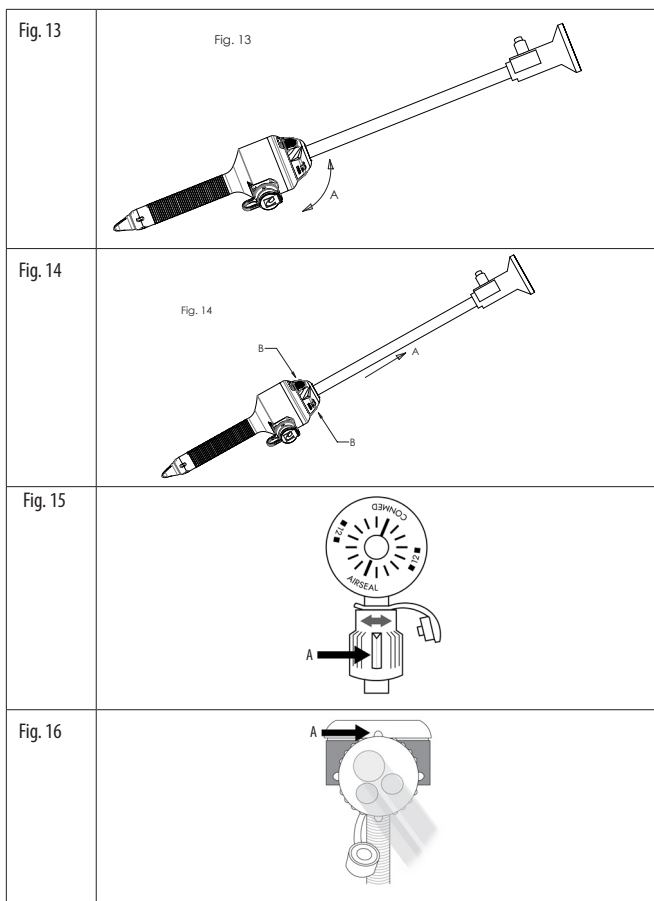**ENGLISH**

**AirSeal® IFS**

**AirSeal® Access Port**

Bladeless Optical Tip, Low Profile Bladeless Optical Tip, Palm Grip Bladeless Optical Tip, and Blunt Tip Access Ports

**AirSeal® Filtered Tube Sets**

Tri-Lumen Filtered Tube Set with Activated Charcoal Filter, Bifurcated Smoke Evac Filtered Tube Set with Activated Charcoal Filter, Single Lumen Filtered Tube Set

**Schematic View: Illustration and Nomenclature**

<p><b>AirSeal IFS (Fig. 1)</b></p> <ul style="list-style-type: none"> <li>A. ON/OFF Switch</li> <li>B. Touch Screen Display</li> <li>C. Receptacle for AirSeal and Smoke Evacuation Modes</li> <li>D. Lever for locking tube set in place</li> <li>E. Insufflation tube connection for Standard Insufflation Mode</li> </ul> <p><b>Bladeless Access Ports (Fig. 2/3)</b></p> <ul style="list-style-type: none"> <li>A. Manifold Plug</li> <li>B. Cannula</li> <li>C. Bladeless Optical Tip Obturator</li> <li>D. Cannula Latch</li> </ul> <p><b>Blunt Tip Access Ports (Fig. 4)</b></p> <ul style="list-style-type: none"> <li>A. Manifold Plug</li> <li>B. Cannula</li> <li>C. Blunt Tip Obturator</li> <li>D. Spring Anchor – Suture Tie Down Anchor Device</li> </ul> <p><b>Optional Sound Cap (Fig. 5)</b></p>	<p><b>Tube Sets (Fig. 6/7/8)</b></p> <ul style="list-style-type: none"> <li>A. Filter</li> <li>B. 8-10 ft. Tube</li> <li>C. Luer Lock Fitting</li> <li>D. Single Lumen Adapter Assembly</li> <li>E. Thumb Collar</li> </ul> <p><b>AirSeal Mode (Fig. 9)</b></p> <p><b>Smoke Evacuation Mode (Fig. 10)</b></p> <p><b>Standard Insufflation Mode (Fig. 11)</b></p> <p><b>Endoscope Insertion (Fig. 12)</b></p> <ul style="list-style-type: none"> <li>A. AirSeal Access Port</li> <li>B. Appropriately Sized Endoscope</li> </ul> <p><b>Endoscope Insertion (Fig. 13)</b></p> <ul style="list-style-type: none"> <li>A. Rotating motion during insertion will ease insertion force</li> </ul> <p><b>Endoscope Removal (Fig. 14)</b></p> <ul style="list-style-type: none"> <li>A. Retract endoscope</li> <li>B. Press here to release cannula</li> </ul> <p><b>Top View of AirSeal Access Port with proper connection to AirSeal Tube Set (Fig. 15)</b></p> <ul style="list-style-type: none"> <li>A. Tube Set Bullseye Connector Fin</li> </ul> <p><b>Front View of AirSeal Access Port with proper connection to AirSeal Tube Set (Fig. 16)</b></p> <ul style="list-style-type: none"> <li>A. Tube Set Bullseye Connector Fin</li> </ul>
--	--

## **IMPORTANT! BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.**

This booklet is designed to assist in using this product. It is not a reference to surgical techniques. In addition, please refer to the Instructions For Use of the AirSeal IFS for full instructions on the AirSeal IFS and all modes of operation.

### **INDICATIONS FOR USE:**

The CONMED AirSeal™ IFS System is intended for use in diagnostic and/or therapeutic endoscopic procedures to distend a cavity by filling it with gas, to create and maintain a gas-sealed obstruction-free path of entry for endoscopic instruments and to evacuate surgical smoke. It is indicated for use in abdominal, thoracic, and pediatric ( $\geq 20$  kg) procedures where insufflation is desired to facilitate the use of various thoracoscopic and laparoscopic instruments by filling the abdominal or thoracic cavity with gas to distend it, by creating and maintaining a gas sealed obstruction-free instrument path and by evacuating surgical smoke. This instrument can also be used to insufflate the rectum and colon to facilitate endoscopic observation, diagnosis, and treatment. The trocar of the AirSeal™ IFS System is indicated for use with or without visualization.

### **The AirSeal IFS has three modes of operation:**

- 1. AirSeal Mode (Fig. 9)**
- 2. Smoke Evacuation Mode (Fig. 10)**
- 3. Standard Insufflation Mode (Fig. 11)**

#### **AirSeal Mode:**

When used in the AirSeal Mode, the AirSeal IFS is designed to provide CO<sub>2</sub> gas delivery with stable pneumoperitoneum and continuous smoke evacuation during laparoscopic surgery and requires the use of the AirSeal IFS, AirSeal Access Port, and Tri-Lumen Filtered Tube Set with Activated Charcoal Filter. The Tri-Lumen Filtered Tube Set with Activated Charcoal Filter is used to connect the AirSeal Access Port to the AirSeal IFS (Fig. 9).

#### **Smoke Evacuation Mode:**

When used in the Smoke Evacuation Mode, the AirSeal IFS is designed to provide CO<sub>2</sub> gas delivery and continuous smoke evacuation during laparoscopic surgery and requires the use of the AirSeal IFS, two conventional (Non-AirSeal) Trocars with standard Luer ports, and the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter. The two conventional trocars are used to connect the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter to provide CO<sub>2</sub> gas delivery along with a Smoke Evacuation pathway. The Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter is bifurcated and connects to the two conventional trocars to the AirSeal IFS (Fig. 10).

#### **Standard Insufflation Mode:**

When used in Standard Insufflation Mode, the AirSeal IFS is designed to provide CO<sub>2</sub> gas delivery and Standard Insufflation during laparoscopic surgery in the same format as conventional insufflators, and requires the use of the AirSeal IFS, one conventional (Non-AirSeal) Trocar with standard Luer port, and the Single Lumen Filtered Tube Set. The Luer port on the trocar connects to the Single Lumen Filtered Tube Set to provide standard insufflation and CO<sub>2</sub> gas delivery. The Single Lumen Filtered Tube Set connects one conventional trocar to the AirSeal IFS (Fig. 11).

### **The AirSeal Access Port has two available configurations:**

- 1. Bladeless Optical Tip Access Port (Figs. 2 & 3)**
- 2. Blunt Tip Access Port (Fig. 4)**

If any of the Bladeless Optical Tip Access Ports are used for primary entry, use of a laparoscope to provide visualization is recommended to enhance safe abdominal entry.

### **The AirSeal Tube Sets have three available configurations:**

- 1. Tri-Lumen Filtered Tube Set with Activated Charcoal Filter (Fig. 6)**
- 2. Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter (Fig. 7)**
- 3. Single Lumen Filtered Tube Set (Fig. 8)**

Ensure that the proper tube set is used based on the mode selected on the AirSeal IFS. The selected AirSeal IFS mode will not proceed, and a warning message will appear, if the wrong tube set is inserted.

The filtered tube sets are not designed to take in fluid. If fluid enters the filter, a warning message will appear on the touch screen display of the AirSeal IFS. If this happens, check cannula placement, and prepare to replace the tube set.

### **WARNINGS:**

- Failure to properly follow the instructions for use can lead to serious surgical consequences.
- Only qualified physicians with knowledge, experience, and training in laparoscopic techniques should use the components of the AirSeal System.
- These instructions for use do not include descriptions or instructions for surgical techniques or laparoscopic procedures. It is the responsibility of the physician performing any procedure to determine the appropriateness of the type of procedure to be performed with the use of these products and to determine the specific technique for each patient.
- The enclosed components of the AirSeal System are only sterile if used before the expiration date and if the package is unopened and undamaged. DO NOT use after the expiration date or if package is open or damaged.
- All single-use AirSeal System devices (Access Ports and Tube Sets) are packaged and sterilized for single patient use only. Do not reuse, reprocess, or resterilize. Reuse, reprocessing, or resterilization may compromise the structural integrity of the device and/or lead to device failure, which in turn may result in patient injury, illness, or death. Also, reprocessing or resterilization of single-use devices may create a risk of contamination and/or patient infection or cross-infection including, but not limited to, the transmission of infectious disease(s) from one patient to another. Device contamination may lead to patient injury, illness, or death.
- Do not submerge the tip of the cannula in irrigation or bodily fluids.
- The AirSeal IFS alarm will sound if the cooling vents are blocked on the sides or the bottom of the control console. If the alarm sounds, check for any blockage of the cooling vents.
- If the above conditions DO NOT exist and the alarm continues to sound, stop using the AirSeal IFS and return to the manufacturer for servicing.

### **PRECAUTIONS:**

- The creation of pneumoperitoneum has known physiologic side effects, including decreased venous return, decreased cardiac output, reduced renal function, increased intracranial pressure, increased airway resistance, and reduced pulmonary compliance.
- Only persons having adequate training, familiarity, and relevant competence with minimally invasive surgical techniques should perform minimally invasive procedures. Consult medical literature relative to techniques, complications, and hazards prior to performance of any minimally invasive procedure.
- A thorough understanding of the principles and techniques involved in laser, electrosurgical, and ultrasonic procedures is essential to avoid shock and burn hazards to both patient and medical personnel and damage to the device or other medical instruments. Ensure that electrical insulation or grounding is not compromised.
- The possibility of air entrapment exists under the following conditions:
  - Severe pressure applied externally to the abdomen
  - Severe and protracted leaking through other conventional ports in place or an open incision
  - Severe and prolonged suction
- During these conditions as described above, the displacement of the insufflation gas with air is temporary and the entrained air will be displaced by CO<sub>2</sub>.
- It is recommended to not exceed insufflation pressures  $>15$  mm Hg in colo-rectal procedures.
- Higher insufflation pressures ( $>15$  mm Hg) of carbon dioxide insufflation can increase the risk of hypercarbia, subcutaneous emphysema, pneumomediastinum, pneumothorax, pneumoscrotum, and urinary retention.
- If using the optional Sound Cap (8 mm, 12 mm):
  - Inspect Sound Cap foam and seal prior to use.
  - Inspect the Sound Cap after use for physical damage of any kind.

- Use extreme caution during AirSeal Access Port insertion. Improper use of this product can result in life-threatening injury to internal organs and vessels.
  - Ensure that adequate pneumoperitoneum or pneumoretum is established
  - Ensure that the patient is properly positioned so that organs are away from the penetration site
  - Direct the AirSeal Access Port's tip away from significant vessels and organs
  - Take care that the outer wall of the AirSeal Access Port is only in contact with tissue and is neither trapped between nor collides with hard surfaces, such as other instruments or bone. Prolonged or sudden contact with hard surfaces can result in damage to or breakage of the Access Port.
  - Do not use excessive downward force
- Only surgical hand instruments should be passed down the AirSeal Access Port. DO NOT insert trocars or robotic instruments down the AirSeal Access Port. Improper use can result in damage to or breakage of the Access Port.
- The design features of the Bladeless Optical Tip Access Port are intended to minimize the likelihood of penetration injury to underlying anatomical structures. However, the standard precautionary measures employed in all such insertions must be observed.
- Although the Bladeless Optical Tip Access Port has a "bladeless" tip, care must still be taken, as with all such devices, to avoid damage to major vessels and other anatomic structures (such as bowel or mesentery). To minimize the risk of such injury, be sure to:
  - Establish adequate pneumoperitoneum or pneumoretum;
  - Properly position the patient to help displace organs out of the area of penetration
  - Note important anatomical landmarks
  - Direct AirSeal Access Port's tip away from major vessels and structures
  - Do not use excessive or uncontrolled force
  - Employ optical entry technique whenever possible.
- Once partial entry has been accomplished, very little force is required to complete entry. Excessive force may cause injury to underlying anatomic structures.
- Once proper entry has been made endoscopically, the Bladeless Optical Tip Access Port should not be advanced for additional penetration. Continued entry of the Access Port at this point could cause injury to underlying anatomic structures.
- Once a safe and proper entry has been achieved, ensure that the black line at the distal tip of the AirSeal Access Port is visible within the cavity at all times the AirSeal Access Port is being used for insufflation.
- Do not attempt secondary trocar punctures until the primary site and recommended intraabdominal pressure (usually 12-18 mm Hg) are established.
- After removing the Cannula from the cavity, always inspect the site for hemostasis. If hemostasis is not present, appropriate techniques should be used to achieve hemostasis.
- The Tri-Lumen Filtered Tube Set with Activated Charcoal Filter and Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter are equipped with fluid sensors to monitor the fluid level in the filters and to warn users of a possible device contamination. If the fluid trap of the filter housing is filled to the **Low** fluid level, a warning message is depicted on the touch screen display and an audible signal is emitted. Cannula and tubing placement should be checked to make sure no more fluid enters the filter. At this point, reinsert obturator into AirSeal Access Port and change the filtered tube set.
- Instruments, or devices, which come into contact with bodily fluids, may require special disposal handling to prevent biological contamination.

#### CONTRAINDICATIONS

This AirSeal IFS is contraindicated for use when endoscopic minimally invasive techniques are contraindicated.

**IMPORTANT:** This package insert is designed to provide instructions for use. It is not a reference or guide for trocar insertion techniques. Successful use of a bladeless optical tip trocar as the primary port after insufflation depends upon recognizing and differentiating between tissue layers. Therefore, new users should utilize the optical tip trocars as a secondary port following insufflation to gain experience and procedural competence in visualizing the tissue layers. Only after achieving experience with the above technique should optical tip trocars be used as a primary port after insufflation.

**Note:** Visually inspect sterile barrier system(s) for any package integrity issues that could impact sterility prior to aseptic presentation. Do not use if any breaches in the sterile barrier system are found.

#### INSTRUCTIONS FOR USE

##### General Instructions for Use of the AirSeal Access Port for AirSeal Mode:

**Note:** Refer to IFS Instructions For Use for features associated with specific software revisions.

1. Turn ON the AirSeal IFS by pressing the ON/OFF Switch (Fig. 1, A).
2. Lever on AirSeal IFS should be in the up or UNLOCK position (Fig. 1, D).
3. Allow the initial self-check to conclude before inserting Tri-Lumen Filtered Tube Set.
4. Using sterile technique, remove the Tri-Lumen Tube Set from the package. To avoid damage, do not flip the device onto the sterile field.
5. Insert filter housing of the Tri-Lumen Tube Set into the Tube Set Receptacle located on the front of AirSeal IFS (Fig. 1, C).
6. Push the lever down into the LOCK position (Fig. 9), AirSeal mode will launch automatically.
 

**Note:** If AirSeal Pediatric Mode is activated, you will be prompted to select Adult or Pediatric AirSeal Mode. Refer to the Instructions For Use for the AirSeal IFS for specifics on Pediatric AirSeal mode.
7. Verify on the AirSeal IFS touch screen display that there is sufficient gas supply level (Fig. 1, B).
8. Select desired flow and pressure settings for the patient. Press "Confirm" to confirm patient settings.

**See further instructions below for Veress Needle Entry with the Bladeless Optical Tip Access Port. This technique is not used for Blunt Tip (Hasson-type) abdominal entry. If Veress needle entry technique is not used, proceed to step 9.**

- The Single Lumen Adapter is pre-attached to the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter.
- Connect the Single Lumen Adapter to the Veress needle Luer port and insert in accordance with the manufacturer's instructions.
- Set desired flow and pressure values on the AirSeal IFS touch screen display.
- Push "Start" on the AirSeal IFS touch screen display, and Initial Insufflation will begin.
- After Initial Insufflation has been achieved, detach the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter from the Single Lumen Adapter.
- Remove the Veress Needle.

**If the AirSeal Access Port is not the first trocar to be placed, keep the Single Lumen Adapter connected to the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter, and transition the Single Lumen Adapter to the Luer port of the first conventional trocar placed. Only remove the Single Lumen Adapter once the AirSeal Access Port has been inserted.**

9. Using sterile technique, remove the AirSeal Access Port and optional Sound Cap (8 mm, 12 mm) from the package. To avoid damage, do not flip the device onto the sterile field.
10. The AirSeal Access Port (obturator and cannula) comes packaged pre-assembled, so there is no need to assemble them. Note: Should the Access Port require assembly, insert the obturator into the cannula until they lock securely together.
11. Ensure that the obturator and cannula are properly assembled. **Note:** If using the Blunt Tip Access Port, ensure that the Spring Anchor is also in place (Fig. 4, D).
12. Create an incision using standard surgical procedures. (Note: Incision should accommodate diameter of cannula. An inadequate incision may cause increased resistance to insertion, increasing the required penetration force, and possibly resulting in a loss of control during entry.)

**The assembled Access Port is ready to be inserted. If using the Blunt Tip Access Port, use appropriate laparoscopic (Hasson-style entry) technique.**

13. Introduce the Access Port assembly through the skin incision using a 30° to 90° rotating motion. Apply continuous but controlled downward pressure on the handle.
14. Ensure that the Access Port is properly inserted. Do not remove obturator from cannula at this time. **Note:** If using the Blunt Tip Access Port, the Spring Anchor should be secured down to the patient's abdomen with suture, in the usual manner.

- Ensure that the black line at the distal tip of the AirSeal Access Port is visible within the cavity at all times that the AirSeal Access Port is being used for insufflation.
- Remove the manifold plug from the side of the cannula on the AirSeal Access Port and connect the distal end of the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter to the cannula manifold and tighten.  
**Note:** If the AirSeal Tube Set and Access Port are not appropriately tightened, the pressure reading on the device may not reflect the pressure experienced by the patient. A secure connection can be visually confirmed when one of the white fins on the Tube Set bullseye connector aligns directly to the top of the Access Port (Fig. 15). Looking straight into the bullseye of the Access Port while the Tube Set is connected, one of the fins on the connector piece of the Tube Set should sit directly vertical to the Access Port (Fig. 16).
- AirSeal will activate automatically. Leave obturator in place until audible tone and green "AirSeal active" message is shown.
- Remove the obturator from the cannula by squeezing on the obturator latch buttons to unlock the two devices. **Note:** The Blunt Tip Access Port does not have latch buttons, so it can be removed by pulling the obturator straight out of the cannula.
- Once AirSeal Mode is active, and the obturator has been removed from the AirSeal cannula, the flow rate will be automatic. **Note:** The level of smoke evacuation can be set to either a High or Low setting.
- If desired, optional Sound Cap (8 mm, 12 mm) may be placed on cannula at this time. (**Note:** If using optional Sound Cap, use caution when inserting a sharp or large device through the cannula. The optional Sound Cap may be removed from the cannula at any time during the procedure and should be removed before inserting any sharp or large objects into the cannula.)
- Upon completion of the procedure, press **STOP** on the touch screen display to stop AirSeal insufflation (**Note:** this will result in loss of pneumoperitoneum). Using standard laparoscopic technique, remove all surgical devices and close applicable port sites in the usual fashion. **Note:** When procedure is completed, AirSeal Mode will undergo a final calibration prior to shut down. During this process no pressure or insufflation is built up. Wait for final calibration to complete prior to removing the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter or powering off the IFS.
- Move the lever on the AirSeal IFS to the up or UNLOCK position.
- Remove the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter from the AirSeal IFS Tube Set Receptacle.
- Because the functional test is performed during initial start up, the unit must be power cycled (off/on) prior to each surgery.

**See further instructions below for the Bladeless Optical Tip Access Port insertion with the use of an Endoscope.**

**Note: Refer to IFS Instructions For Use for features associated with specific software revisions.**

- Connect the 0° endoscope to the light source and monitor as directed in the manufacturer's instructions. Verify proper connection of the endoscope and ensure clarity of the picture on the monitor.
- Insert the endoscope into the opening at the proximal end of the Bladeless Optical Tip Access Port handle until it reaches the distal tip of the Access Port. (Endoscope Insertion; Fig. 12).
- Rotate the endoscope as desired. (Endoscope Insertion; Fig. 13).
- To provide a clear image on the monitor, insert the endoscope into the Bladeless Optical Tip Access Port handle, touch the optical tip to a convenient soft surface, and focus the camera.
- Create an incision using standard surgical procedures. (**Note:** Incision should accommodate diameter of Cannula. An inadequate incision may cause increased resistance to insertion, increasing the required penetration force, and possibly resulting in a loss of control during entry.) Introduce the Access Port assembly through the skin incision using a 30° to 90° rotating motion. Apply continuous but controlled downward pressure on the handle.
- During insertion of the Access Port, view the penetration of the Access Port through the individual tissue planes by using the laparoscope. The individual tissue planes may be seen as the Access Port advances.
- Remove the laparoscope from the Bladeless Optical Tip Access Port (Endoscope Removal; Fig. 14).
- Continue procedure in accordance with instructions above, beginning with step 13.

**INSTRUCTIONS FOR USE**

**Instructions For Use of the AirSeal Access Port for AirSeal Mode in Trans-Anal Applications:**

**Note: Refer to IFS Instructions For Use for features associated with specific software revisions.**

- Turn ON the AirSeal IFS by pressing the ON/OFF Switch (Fig. 1, A).
- Lever on AirSeal IFS should be in the up or UNLOCK position (Fig. 1, D).
- Allow the initial self-check to conclude before inserting Tri-Lumen Filtered Tube Set.
- Using sterile technique, remove the Tri-Lumen Tube Set from the package. To avoid damage, do not flip the device onto the sterile field.
- Insert filter housing of the Tri-Lumen Tube Set into the Tube Set Receptacle located on the front of AirSeal IFS (Fig. 1, C).
- Push the lever down into the LOCK position (Fig. 9), AirSeal mode will launch automatically.  
**Note:** If AirSeal Pediatric Mode is activated, you will be prompted to select Adult or Pediatric AirSeal Mode. Refer to the Instructions For Use for the AirSeal IFS for specifics on Pediatric AirSeal mode.
- Verify on the AirSeal IFS touch screen display that there is sufficient gas supply level (Fig. 1, B).
- Select desired flow and pressure settings for the patient. Press "Confirm" to confirm patient settings.
- Using sterile technique, remove the AirSeal Access Port and optional Sound Cap (8 mm, 12 mm) from the package. To avoid damage, do not flip the device onto the sterile field.
- The AirSeal Access Port (obturator and cannula) comes packaged pre-assembled, so there is no need to assemble them.  
**Note:** Should the Access Port require assembly, insert the obturator into the cannula until they lock securely together.
- Ensure that the obturator and cannula are properly assembled.
- Follow the instructions for use provided by the manufacturer of chosen trans-anal access platform device for placement into the rectum.
- Insert the Access Port into the trans-anal access platform device through the appropriate lumen or cannula position. When inserting the Access Port into the trans-anal access platform device, introduce using a 30° to 90° rotating motion. Apply continuous but controlled axial pressure on the Access Port until secure in the trans-anal access platform device.
- Ensure that the obturator and cannula are properly inserted. Do not remove obturator from cannula assembly at this time.
- Remove the manifold plug from the side of the cannula on the AirSeal Access Port and connect the distal end of the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter to the cannula manifold and tighten.  
**Note:** If the AirSeal Tube Set and Access Port are not appropriately tightened, the pressure reading on the device may not reflect the pressure experienced by the patient. A secure connection can be visually confirmed when one of the white fins on the Tube Set bullseye connector aligns directly to the top of the Access Port (Fig. 15). Looking straight into the bullseye of the Access Port while the Tube Set is connected, one of the fins on the connector piece of the Tube Set should sit directly vertical to the Access Port (Fig. 16).
- AirSeal will activate automatically. Leave obturator in place until audible tone and green "AirSeal active" message is shown.
- Remove the obturator from the cannula by squeezing on the obturator latch buttons to unlock the two devices.
- Once AirSeal Mode is active, and the obturator has been removed from the AirSeal cannula, the flow rate will be automatic.  
**Note:** The level of smoke evacuation can be set to either a High or Low setting.
- If desired, optional Sound Cap (8 mm, 12 mm) may be placed on cannula at this time. (**Note:** If using optional Sound Cap, use caution when inserting a sharp or large device through the cannula. The optional Sound Cap may be removed from the cannula at any time during the procedure and should be removed before inserting any sharp or large object into the cannula.)
- Upon completion of the procedure, press **STOP** on the touch screen display to stop AirSeal insufflation (**Note:** this will result in loss of pneumoperitoneum). Using standard laparoscopic technique, remove all surgical devices and close applicable port sites in the usual fashion.  
**Note:** When procedure is completed, AirSeal Mode will undergo a final calibration prior to shut down. During this process no pressure or insufflation is built up. Wait for final calibration to complete prior to removing the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter or powering off the IFS.
- Move the lever on the AirSeal IFS to the up or UNLOCK position.
- Remove the Tri-Lumen Filtered Tube Set with Activated Charcoal Filter from the AirSeal IFS Tube Set Receptacle.
- Because the functional test is performed during initial start up, the unit must be power cycled (off/on) prior to each surgery.

## General Instructions for Use in - Smoke Evacuation and Standard Insufflation Modes:

### INITIAL INSUFFLATION WITH VERESS NEEDLE

**Note: Refer to iFS Instructions For Use for features associated with specific software revisions.**

1. Turn ON the AirSeal iFS by pressing the ON/OFF Switch (Fig. 1, A).
2. At conclusion of diagnostic self-check, select either Smoke Evacuation Mode, or Standard Insufflation Mode on the touch screen display, from the mode selection list on the AirSeal iFS (Fig. 1, B).
3. Verify on the AirSeal iFS touch screen display that there is sufficient gas supply level (Fig. 1, B).
4. Connect appropriate tube set to the AirSeal iFS (See specific instructions below for connecting the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter).  
**Note:** Unlike the other tube sets, the single lumen tube set connects to the insufflation tube connecting post on the front of the AirSeal iFS.
5. Connect the Luer lock connector of the tube set to the end of a Veress needle or conventional Trocar in the usual manner. (See specific instructions below for connecting the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter).
6. Set desired flow and pressure values on the AirSeal iFS touch screen display. Press "Confirm" to confirm patient settings.
7. Push "Start" on the AirSeal iFS touch screen display, and Initial Insufflation will begin.
8. After Initial Insufflation has been achieved **insert the conventional Trocar with standard Luer ports.**
9. Upon completion of the procedure, press **STOP** on the touch screen display to stop insufflation (**Note:** this will result in loss of pneumoperitoneum). Using standard laparoscopic technique, remove all surgical devices and close applicable port sites in the usual fashion.
10. Remove the Tube Set from the AirSeal iFS (See specific instructions below for removal of the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter).
11. Because the functional test is performed during initial start up, the unit must be power cycled (off/on) prior to each surgery.

### See further instructions below for the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter

#### For use with two conventional trocars with standard Luer ports and the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter.

Clear Tube: Insufflation

Blue Tube: Smoke Evacuation

#### Attaching the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter

**Note: Refer to iFS Instructions For Use for features associated with specific software revisions.**

1. Lever on AirSeal iFS should be in the up or UNLOCK position (Fig. 1, D).
2. Insert filter housing of the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter into the Tube Set Receptacle located on the front of the AirSeal iFS (Fig. 1, C).
3. Push the lever down into the LOCK position.
4. Attach Veress needle to Luer lock connector of the clear tube on the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter.
5. Set desired flow and pressure values on the AirSeal iFS touch screen display. Press "Confirm" to confirm patient settings.
6. Push "Start" on the AirSeal iFS touch screen display and Initial Insufflation will begin.
7. Ensure that the cannulas are properly inserted.
8. Remove Veress needle and disconnect from Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter.
9. Connect the Luer lock connector of the **CLEAR** tube of the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter to the conventional cannula. Open the inflow valve/stopcock.
10. Follow the prompts on the AirSeal iFS touch screen display. When instructed, connect the Luer lock connector of the **BLUE** tube of the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter to a different conventional cannula. Insufflation remains active.
11. When the Luer lock of the connector of the **BLUE** tube Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter is connected to the second conventional cannula, Smoke Evacuation is activated.

**SERIOUS INCIDENTS - Immediately report any serious incidents with the use of this device to the CONMED Customer Service Department or your local CONMED representative. Email: CustomerExperience@CONMED.com**

#### Removal of the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter

**Note: Refer to iFS Instructions For Use for features associated with specific software revisions.**

1. Upon completion of the procedure, press **STOP** on the touch screen display to stop insufflation (**Note:** this will result in loss of pneumoperitoneum). Using standard laparoscopic technique, remove all surgical devices and close applicable port sites in the usual fashion.
2. Move lever on the AirSeal iFS to the up or UNLOCK position.
3. Remove the Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter from the AirSeal iFS Tube Set Receptacle.
4. Because the functional test is performed during initial start up, the unit must be power cycled (off/on) prior to each surgery.

### DISPOSAL

Dispose of the device in accordance with local hospital requirements.

### HOW SUPPLIED

Each of the AirSeal Optical Access Port, AirSeal Low Profile Access Port, Optional Sound Cap (8 mm, 12 mm), AirSeal Blunt Tip Access Port, AirSeal Cannula, Bifurcated Smoke Evac Tube Set with Activated Charcoal Filter, Tri-Lumen Filtered Tube Set with Activated Charcoal Filter, Single Lumen Tube Set, and AirSeal Single Lumen Adapter, are supplied sterile through gamma radiation for single patient use. Discard after use. The AirSeal Access Ports, Sound Caps, and Tube Sets are not made with Natural Rubber Latex.

The AirSeal iFS is non-sterile and reusable.

AirSeal® is a registered trademark of CONMED Corporation.

**Rx ONLY** CAUTION: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

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All rights reserved.
MDSS GmbH
©2023 CONMED Corporation.